OCULOPLASTIK

(en) Surgical instruments.....	1
(fr) Instruments chirurgicaux.....	3
(de) Chirurgische Instrumente.....	5
(es) Instrumentos quirúrgicos.....	7
(it) Strumenti chirurgici.....	9
(nl) Chirurgische instrumenten.....	11
(pt) Instrumentos cirúrgicos.....	14

(en) SURGICAL INSTRUMENTS

INSTRUCTIONS FOR USE

READ ALL INSTRUCTIONS BEFORE USING THIS PRODUCT.

CAUTION, consult accompanying documents.

This Product is sold Non Sterile.

oculoplastik.com

Get our instructions for use at oculoplastik.com.

DECONTAMINATION

- Immerse the instruments in a decontaminating solution and follow the manufacturer's instruction.
- Avoid the use of chlorine that may corrode the stainless steel instruments.

CLEANING PROCEDURE FOR THE DEVICES

Devices must be cleaned before sterilization. Use procedures that have been previously established and validated for your institution or use the following recommendations.

1. Mild soap solution and soft bristled toothbrush
2. Ultrasonic procedure for at least 5 minutes
3. Using a washing machine
4. Dry instruments quickly and completely
5. Rinse extensively with distilled or demineralized water after cleaning.

STERILIZATION RECOMMENDATIONS

- Stainless steel instruments can be sterilized repeatedly by steam autoclave.
- Do not sterilize stainless steel devices in bleach, as bleach will corrode them.
- Treat the instruments made from different metals separately.

- Do not mix instruments in good condition with instruments that are corroded or damaged.

STEAM AUTOCLAVE

Steam sterilize following a validated cycle according to your institution's standards OR the following recommended parameters.

Local or national specifications should be followed where steam sterilization requirements are stricter or more conservative than those listed in the following recommended parameters.

Cycle Type	Temperature	Exposure Time	Dry Time
UK Prevacuum	134°C (273°F)	3 minutes	30 minutes
Prevacuum	132°C (270°F)	4 minutes	30 minutes

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ATTENTION, consulter les documents joints.

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OCULO-PLASTIK, Inc.