
LASERSECURE

(en) OPSoft Mouthguards	1
(fr) Protecteurs buccaux OPSoft	2
(de) OPSoft-Mundschutz	3
(es) Protectores bucales OPSoft	4
(it) Protettore dentale OPSoft	5
(nl) OPSoft-Mondbescherming.....	6
(pt) Protetores para a boca OPSoft.....	7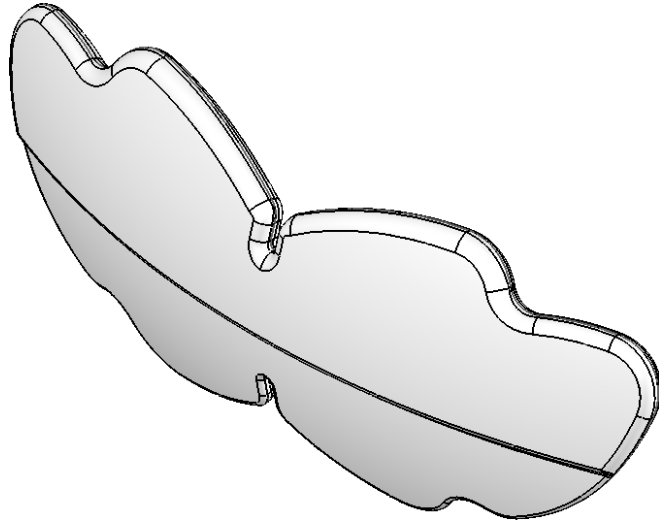

(en) OPSOFT MOUTHGUARDS

INSTRUCTIONS FOR USE

READ ALL INSTRUCTIONS BEFORE USING THIS PRODUCT.

CAUTION, consult accompanying documents.

oculoplastik.com

Get our instructions for use at oculoplastik.com.

This product is latex free.

This Product is sold Non Sterile.

Do not autoclave.

Do not reuse.

The OPSoft mouthguard is intended for protection during hair removal, cutaneous treatment, and perioral procedures with laser, IPL or RF.

This medical device is made of soft thermoplastic that disperses the energy and avoids penetration. The OPSoft mouthguard is inserted between the lips and teeth by the physician, or his assistant. It eliminates pain and protects the teeth & gums. The OPSoft mouthguard easily conforms to the buccal area. It may be used over dental braces, repairs and

implants. Mouthguard may be cut smaller (if required) to fit patient.

WARNING

This shield cannot be autoclaved. The material will melt.

Must not touch any acids, oxidizers or hard reducing agent, because materials will decompose.

The OPSOFT mouthguard is a **single use** medical device for a **single patient**:

- A second use of OPSOFT mouthguard at a later time may cause an infection due to microbial contamination.

OPSOFT Mouthguard may be used with all lasers and Intense Pulse Light (IPL) as well as with RF units.

Spot test the OPSOFT mouthguard with any new energy device before use.

Do not use with CO2 laser.

The OPSOFT mouthguard is not intended for any other use except the use recommended by Oculo-Plastik, inc.

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ATTENTION, consulter les documents joints.

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O protetor para a boca OPSOFT somente deverá ser usado de acordo com as recomendações de Oculo-Plastik, Inc.