
OCULOPLASTIK

(en) Suction Cups	1
(fr) Ventouses	2
(de) Saugnapfen	4
(es) Ventosas	5
(it) Ventose	7
(nl) zuignappen	8
(pt) Coberturas de sucção.....	10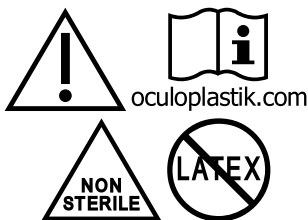

(en) SUCTION CUPS

INSTRUCTIONS FOR USE

READ ALL INSTRUCTIONS BEFORE USING THIS PRODUCT.

CAUTION, consult accompanying documents.

This Product is sold Non Sterile.

Does not contain Natural Rubber (Latex)

oculoplastik.com

Get our instructions for use at oculoplastik.com.

CLEANING PROCEDURE FOR THE DEVICES

Use procedures that have been previously established and validated for your facility or use the following recommendations.

In considering methods for sterilization procedures, it is important to differentiate between sterilization and disinfection. Disinfection only reduces the number of

viable microorganisms. Sterilization kills all viable microorganisms.

- Do not use alcohol as it may damage the devices
1. Before sterilizing, devices should be rinsed under warm running water to remove all debris.
 2. Wipe devices with moist sponge or gauze pad to eliminate any debris.
 3. Devices should be washed in a solution of water and neutral pH pre-soak detergent. DO NOT USE: STAIN REMOVERS, ENZYME CLEANING AGENTS, OR CLEANING AGENTS THAT CONTAIN ALKALIES.
 4. Devices should be cleaned, by hand, ultrasonic cleaner, or automatic washer sterilizer.
 5. Rinse extensively with distilled water after manual cleaning and ultrasonic cleaning. For automatic washer sterilizer follow manufacturer's recommendations.
 6. Sterilize according to manufacturer's instructions.

STERILIZATION RECOMMENDATIONS

Cold sterilization is not recommended. It will damage the suction cups.

Steam autoclave

Steam sterilize following a validated cycle according to your institution's standards OR the following recommended parameters.

Local or national specifications should be followed where steam sterilization requirements are stricter or more conservative than those listed in the following recommended parameters.

<i>Cycle Type</i>	<i>Temperature</i>	<i>Exposure Time</i>	<i>Dry Time</i>
UK Prevacuum	134°C (273°F)	3 minutes	30 minutes
Prevacuum	132°C (270°F)	4 minutes	30 minutes

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ATTENTION, consulter les documents joints.

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