oculoplastik.com

(en) Aestek® Plastic Instruments .....	1
(fr) Instruments en plastique Aestek® .....	4
(de) Die Aestek® Kunststoffinstrumente.....	6
(es) Instrumentos plásticos Aestek® .....	9
(it) Strumenti in plastica Aestek® .....	12
(nl) Aestek® -kunststof instrumenten.....	14
(pt) Instrumentos Plásticos Aestek® .....	17

(en) AESTEK® PLASTIC INSTRUMENTS

INSTRUCTIONS FOR USE

READ ALL INSTRUCTIONS BEFORE USING THIS PRODUCT.

CAUTION, consult accompanying documents.

This Product is sold Non Sterile.

oculoplastik.com

Get our instructions for use at oculoplastik.com.

WARNING

Do not use any device that is scratched or damaged.

After every case application and before sterilization, always verify visually and manually the condition of the surfaces and edges of the device.

Check surfaces of device for scratches or nicks, discard if necessary.

## CLEANING PROCEDURE FOR THE DEVICES

Devices must be inspected and cleaned before initial sterilization, and subsequently between each patient use.

Use procedures that have been previously established and validated for your facility or use the following recommendations.

In considering methods for sterilization procedures, it is important to differentiate between sterilization and disinfection. Disinfection only reduces the number of viable microorganisms. Sterilization kills all viable microorganisms.

- Do not use a brush or any abrasive pads or cleaning agents as they may damage devices.
  - Do not use alcohol as it may damage the devices
1. Before sterilizing, devices should be rinsed under warm running water to remove all debris.
  2. Wipe devices with moist sponge or gauze pad to eliminate any debris.
  3. Devices should be washed in a solution of water and neutral pH pre-soak detergent. DO NOT USE: CHLORIDE, STAIN REMOVERS, CHLORINE BLEACH, ENZYME CLEANING AGENTS, OR CLEANING AGENTS THAT CONTAIN ALKALIES.
  4. Devices should be cleaned, by hand, ultrasonic cleaner, or automatic washer sterilizer.
  5. Rinse extensively with distilled water after manual cleaning and ultrasonic cleaning. For automatic washer sterilizer follow manufacturer's recommendations.

6. Sterilize according to manufacturer's instructions.

### STERILIZATION RECOMMENDATIONS

Aestek® Plastic instruments are autoclavable. They MAY be steam autoclaved.

Cold sterilization is not recommended. It may damage the plastic.

Steam sterilize following a validated cycle according to your institution's standards OR the following recommended parameters.

Local or national specifications should be followed where steam sterilization requirements are stricter or more conservative than those listed in the following recommended parameters.

<i>Cycle Type</i>	<i>Temperature</i>	<i>Exposure Time</i>	<i>Dry Time</i>
UK Prevacuum	134°C (273°F)	3 minutes	30 minutes
Prevacuum	132°C (270°F)	4 minutes	30 minutes

	<i>Ends Width</i>	<i>Length</i>	<i>Color</i>
Jaeger (more concave) 23-293B, 23-293Y	22 & 24 mm	111 mm	Black or Tr-Yellow
Jaeger (flatter) 23-295B, 23-295Y	24 & 26 mm		
Desmarres 23-311Y	10 & 12 mm	96 mm	Tr-Yellow
Desmarres 23-315Y	14 & 16 mm		

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ATTENTION, consulter les documents joints.

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**OCULO-PLASTIK, Inc.**