(en) Dubois Twistripper 1
(fr) Dubois Twistripper 4

(en) DUBOIS TWISTRIPPER

INSTRUCTIONS FOR USE

READ ALL INSTRUCTIONS BEFORE USING THIS PRODUCT.

CAUTION, consult accompanying documents.

This Product is sold Non Sterile.

oculoplastik.com

Get our instructions for use at oculoplastik.com.

INTENDED USE

The Dubois Twistripper is a surgical instrument to facilitate stripping some types of varicose veins, for a better esthetic result than traditional hooks. This instrument improves the current usual technique. Long varicose veins are fragile and have much adherences. The Dubois Twistripper is rolled around the

vein and slowly pushed in, always rotating and securely following the vein. Once adhesions are gone, it is easy to pull the liberated vein.

WARNING

Do not warp or bend the Dubois Twistripper: it will be damaged and become unusable.

DECONTAMINATION

- Immerse the instruments in a decontaminating solution and follow the manufacturer's instruction.
- Avoid the use of chlorine that may corrode the stainless steel instruments.

CLEANING PROCEDURE FOR THE DEVICES

Devices must be cleaned before sterilization. Use procedures that have been previously established and validated for your institution or use the following recommendations.

1. Mild soap solution and soft bristled toothbrush
2. Ultrasonic procedure for at least 5 minutes
3. Using a washing machine
4. Dry instruments quickly and completely
5. Rinse extensively with distilled or demineralized water after cleaning.

STERILIZATION RECOMMENDATIONS

- Stainless steel instruments can be sterilized repeatedly by steam autoclave.
- Do not sterilize stainless steel devices in bleach, as bleach will corrode them.
- Treat the instruments made from different metals separately.
- Do not mix instruments in good condition with instruments that are corroded or damaged.

STEAM AUTOCLAVE

Steam sterilize following a validated cycle according to your institution's standards OR the following recommended parameters.

Local or national specifications should be followed where steam sterilization requirements are stricter or more conservative than those listed in the following recommended parameters.

Cycle Type	Temperature	Exposure Time	Dry Time
UK Prevacuum	134°C (273°F)	3 minutes	30 minutes
Prevacuum	132°C (270°F)	4 minutes	30 minutes

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ATTENTION, consulter les documents joints.

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