(en) Durette® Plastic External Shields 1

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(en) DURETTE® PLASTIC EXTERNAL SHIELDS

INSTRUCTIONS FOR USE

READ ALL INSTRUCTIONS BEFORE USING THIS PRODUCT.

CAUTION, consult accompanying documents.

oculoplastik.com

Get our instructions for use at oculoplastik.com.

THE PLASTIC EYE CUPS AND PLASTIC TEMPORAL AND NASAL PIECES CANNOT BE AUTOCLAVED.

This Product is sold Non Sterile.

This Product (elastic) Contains Dry Natural Rubber (latex).

The Durette® Plastic External Shields are intended to protect patient’s eyes from apparatuses emitting different wavelengths of non coherent light. They should not be used with Lasers THAT IRRADIATE A COHERENT LIGHT BEAM (spectral irradiance of any wavelength) OR ANY FLASH LAMP OR IPL SYSTEM THAT operates in burst modes that comprise multiple trains of pulses.

THEY SHOULD NOT BE USED with high-power, spectral irradiance in the UVB and UVA wavebands or any other spectral irradiance energy or light source.

THE DURETTE® PLASTIC EXTERNAL SHIELDS can be used with:

- Light emitted by fluorescent lamps, bright LEDs or dichroic filament bulbs.
- VISIBLE VIOLET LIGHT IN THE RANGE OF 405-420.
- Light therapy for skin conditions like psoriasis or eczema which use UVA or UVB PHOTOTHERAPY.

- PUVA TREATMENT.
- LIGHT THERAPY by Red and/or near infrared (NIR) LEDs.
- Photodynamic Therapy (PDT) light sources.
- Narrowband Light emitting Diodes (LEDs) - Optimised, via a matrix of LEDs.

USE THE COLOR BLACK OR WHITE ACCORDING TO THE OUTPUT WAVELENGTH / WAVEBAND OF YOUR MACHINE. THE BLACK PLASTIC EXTERNAL SHIELDS PROVIDE A WIDER RANGE OF COVERAGE THAN THE WHITE.

For BLACK Plastic External Shields:

- OD +5 @ 190-2575 nm
- OD +4 @ 2576-3000 nm

For WHITE Plastic External Shields:

- OD +5 @ 190-590 nm
- OD +4 @ 591-682 nm

After every case application and before cleaning, always verify visually and manually the condition of all surfaces of the PLASTIC External Shields, of the nasal and temporal attachments and of the elastic headband.

Before each use, make sure that the plastic attachments are properly inserted onto the ball on the PLASTIC eye cups (fig. 4). Verify that they swivel loosely without force. Never use forceps, pliers or any type of tools to attach or remove the attachments (fig. 5).

The nasal and temporal plastic attachments require a slight force during assembly to insert and remove from the ball on the eye shields and should be well fastened to the eye shields before each use.

To prevent entrance of light the PLASTIC External Shields must be POSITIONED securely in place over the eyes during a procedure.

WARNING

- Do not use any device that is scratched or damaged.
- Any scratch or damage on the surface of the device reduces the optical density (OD) and renders it LESS EFFECTIVE TO PROTECT AGAINST THE RECOMMENDED WAVELENGTHS.
- The plastic external shields AND temporal and nasal pieces must be inspected after every procedure for cracks, change of color or any damage to the plastic from the light sources. REPEATED USAGE OF THE PLASTIC EXTERNAL SHIELDS and plastic attachments WILL EVENTUALLY damage all plastic. Dispose of them immediately and replace them with new ones if any visible changes are apparent.
- Damaged plastic external shields and plastic attachments may cause breakage and dislodging of the shields during a procedure exposing the eyes to potential dangerous lights, or energies. Accidental exposure of the eyes during a procedure may result in serious personal injury, or blindness.
- Verify that the Durette® PLASTIC external shields are completely covering the eyelids by adjusting the elastic headband attached on the plastic temporals or on the hooks on the eye-cups themselves (fig. 2 and 3).

- DO NOT USE WITH LASERS THAT IRRADIATE A COHERENT LIGHT BEAM (spectral irradiance of any wavelength) OR ANY FLASH LAMP OR IPL SYSTEM THAT operates in burst modes that comprise multiple trains of pulses. THE PLASTIC WILL MELT causing burns to the surface of the skin AND INJURY TO THE EYES. Accidental exposure of the eyes to LASERS OR IPLS may result in serious personal injury, or blindness.
- Always transport the Durette® PLASTIC External Shields in a hard box such as the ONE provided.

After the placement and positioning of the PLASTIC external shields over the lids and before any procedure the patient can verify and confirm to the MEDICAL PROFESSIONAL that there is no entrance of ambient light. The patient must keep the eyes closed at all times during the procedure.

WARNING: DO NOT USE WITH LASERS THAT IRRADIATE A COHERENT LIGHT BEAM OR ANY IPL OR FLASH LAMP SYSTEM. THE PLASTIC WILL MELT CAUSING SERIOUS PERSONAL INJURY, AND/OR BLINDNESS.

The elastic headband must always be adjusted by sliding the cordlock in order to ensure a comfortable and secure fit (fig. 1). The adjusting of the cordlock renders the eyewear tighter or looser according to the need of each patient in order to ensure there is no entrance of light. If the elastic headband is too tight, the nasal area may open allowing entrance of light and may also render the eyeshields uncomfortable. If the elastic headband is too loose the eye shields may not block all light from the edges. Before every procedure the eye shields must be properly positioned on the patient and the elastic headband must be well

adjusted. The patient should open his/her eyes to verify for any entrance of light. The patient must close his/her eyes before the procedure begins.

CLEANING PROCEDURE FOR THE DEVICES

The PLASTIC eye CUPS AND PLASTIC nasal and temporal attachments must be inspected and cleaned WITH ANTIBACTERIAL SOAP AND WATER AND STERILIZED IN A STERILIZING SOLUTION between each patient use.

THE ELASTIC HEADBAND must be inspected and cleaned WITH ANTIBACTERIAL SOAP AND WATER AND STEAM AUTOCLAVED. (DO NOT SOAK THE HEAD BAND IN ANY SOLUTION)

Do not use alcohol ON THE PLASTIC OR HEAD BAND.

DO NOT USE: CHLORIDE, STAIN REMOVERS, CHLORINE BLEACH, ENZYME CLEANING AGENTS, OR CLEANING AGENTS THAT CONTAIN ALKALIES.

STERILIZATION RECOMMENDATIONS

The PLASTIC eye cups can be STERILIZED by soaking in a STERILIZING solution. The elastic head band should not be soaked in any solution but can be STEAM autoclaved.

The plastic temporal and nasal pieces can be sterilized by soaking in a sterilizing solution.

Do not sterilize THE PLASTIC devices in bleach. Exposure to bleach will DAMAGE THEM.

Before sterilizing the PLASTIC external shields IN A SOAKING SOLUTION, separate the ELASTIC HEADBAND attachments from

the PLASTIC shields. To sterilize the ELASTIC HEADBANDS IN THE STEAM AUTOCLAVE, place them in a peel-pouch, on the sterilizer rack, in a single layer. Follow your specific sterilizer equipment manufacturer's recommendations, or use cycles that have been previously established and validated for your facility. (ONLY FOR THE ELASTIC HEADBANDS).

To sterilize the plastic temporal and nasal pieces follow the instructions of the sterilizing solution manufacturer.

AFTER REPEATED SOAKING IN STERILIZING SOLUTIONS THE PLASTIC EYE CUPS AND PLASTIC ATTACHMENTS WILL eventually DAMAGE AND MUST BE DISPOSED OF. (Cycle of damage will vary according to different sterilizing solutions).

Please do not return any product without a return authorization number. (Shipping and handling is the responsibility of the sender).

Call or fax for an RA number and for further details.

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ATTENTION, consulter les documents joints.

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